技术规格/Technical Data Sheet

E-40R HF XD55A L

MONOBLOCK E-40R HF XD55A L

编码 / CODE: **30.04.003.001**

S/N: **HF**

IEC601-1: **Class I Type B**

XD55A X-RAY TUBE

编码 / CODE: **61.18.014.003**

S/N:

旋转阳极 / ROTATING ANODE: **3000RPM**

小焦点 / SMALL FILAMENT: **0.3 mm**

大焦点 / LARGE FILAMENT: **0.6 mm**

灯丝电源 / FILAMENT POWER SUPPLY: **400 Hz**

注意/ CAUTION

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WHEN NOT USED, AFTER 3 MONTH PERIOD, PROCEED TO INSERT RESTORATION FOLLOWING THIS CHART:

1- FLUORO: FROM 60 kV to 120kV(step 10kV), 2 mA, 30 SEC ON + 10 SEC OFF; PAUSE 5 MIN

2- 90 kV, 4 mA, 60 sec. ON + 5 min. OFF; 120 kV, 4 mA, 60 SEC ON + 5 min OFF

IF INSTABILITIES OCCUR, PLEASE STOP THE PROCESS, WAIT 5 MINUTES AND RESUME THE PROCEDURE FROM THE BEGINNING.

CE 0051

IMD CHINA CO., LTD.

1 -2/F, BUILDING 12, No. 28 YUHUA RD. AREA B, TIANZHU AIRPORT INDUSTRIAL ZONE, BEIJING, CHINA

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X射线管组件性能 / PERFORMANCE OF MONOBLOCK

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典型电气参数 / TYPICAL ELECTRICAL CHARACTERISTICS

主绕组到公共端 / PRINCIPAL TO COMMON	25 Ω
副绕组到公共端 / AUXILIARY TO COMMON	50 Ω
主绕组到副绕组 / PRINCIPAL TO AUXILIARY	75 Ω

高压变压器电源 / H.V. TRANSFORMER POWER SUPPLY

IMD逆变器E-9040-5 HF/ IMD INVERTER E-9040-5 HF

工作频率 / OPERATING FREQUENCY	20 kHz
最高峰值输入电压 (X和Y之间) MAXIMUM INPUT PEAK VOLTAGE (X AND Y CONNECTIONS)	350 V
最大峰值输入电流 (约为) MAXIMUM INPUT PEAK CURRENT (APPROX.)	130 A

输出信号 / OUTPUT SIGNALS

kV+ (模拟信号 / ANALOG SIGNAL)	0 $\div$ 6,0 V
kV- (模拟信号 / ANALOG SIGNAL)	0 $\div$ 6,0 V
mA+ AND mA- (模拟信号 / ANALOG SIGNAL)	0 $\div$ 100 mA

热保护 - 常闭 - (逻辑信号) THERMAL SECURITY - NORMALLY CLOSED - (LOGICAL SIGNAL)

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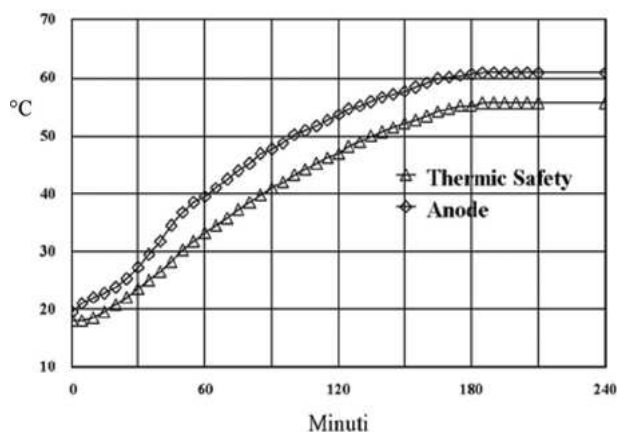
LESS THAN 1 mGy IN ONE HOUR (120 kV-4mA) IN FLUOROSCOPY MODE
ACCORDING TO IEC 601-1-3 APPLICABLE STANDARD (PAR. 29.204)

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
HEATING CURVE (60 kV, 1 mA)

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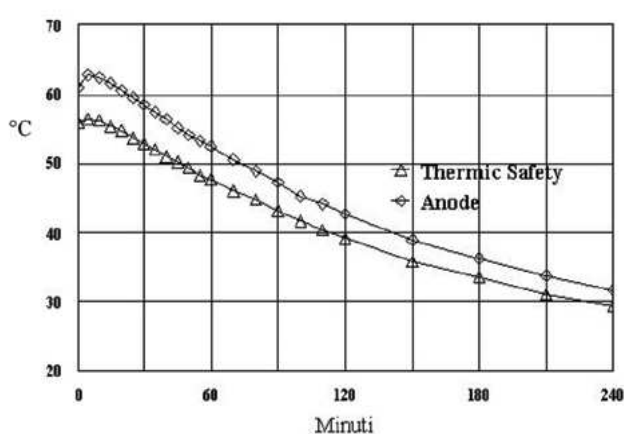COOLING CURVE

IMD CHINA CO., LTD. • 1-2/F, BUILDING 12, No. 28 YUHUA RD. AREA B, TIANZHU AIRPORT INDUSTRIAL ZONE, BEIJING, CHINA

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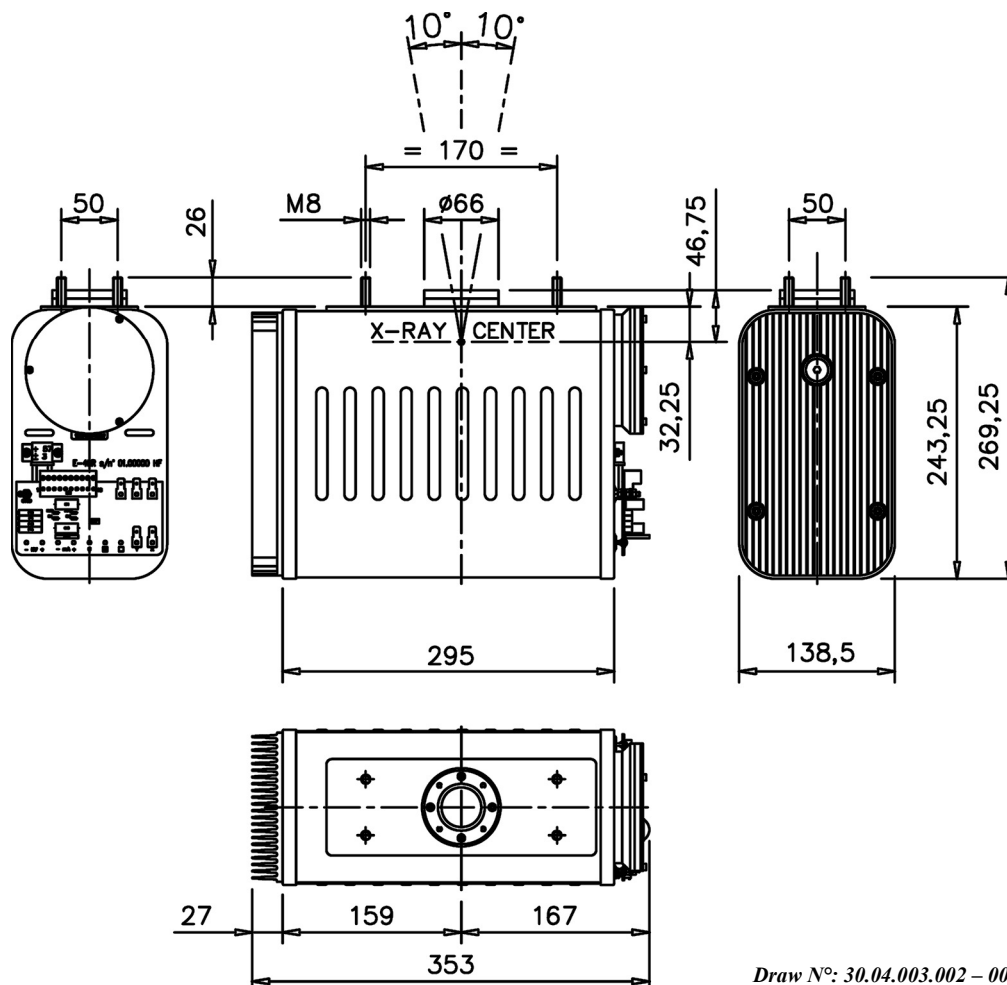
FOR INFORMATION CF. TUBE TECHNICAL DATA SHEET

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
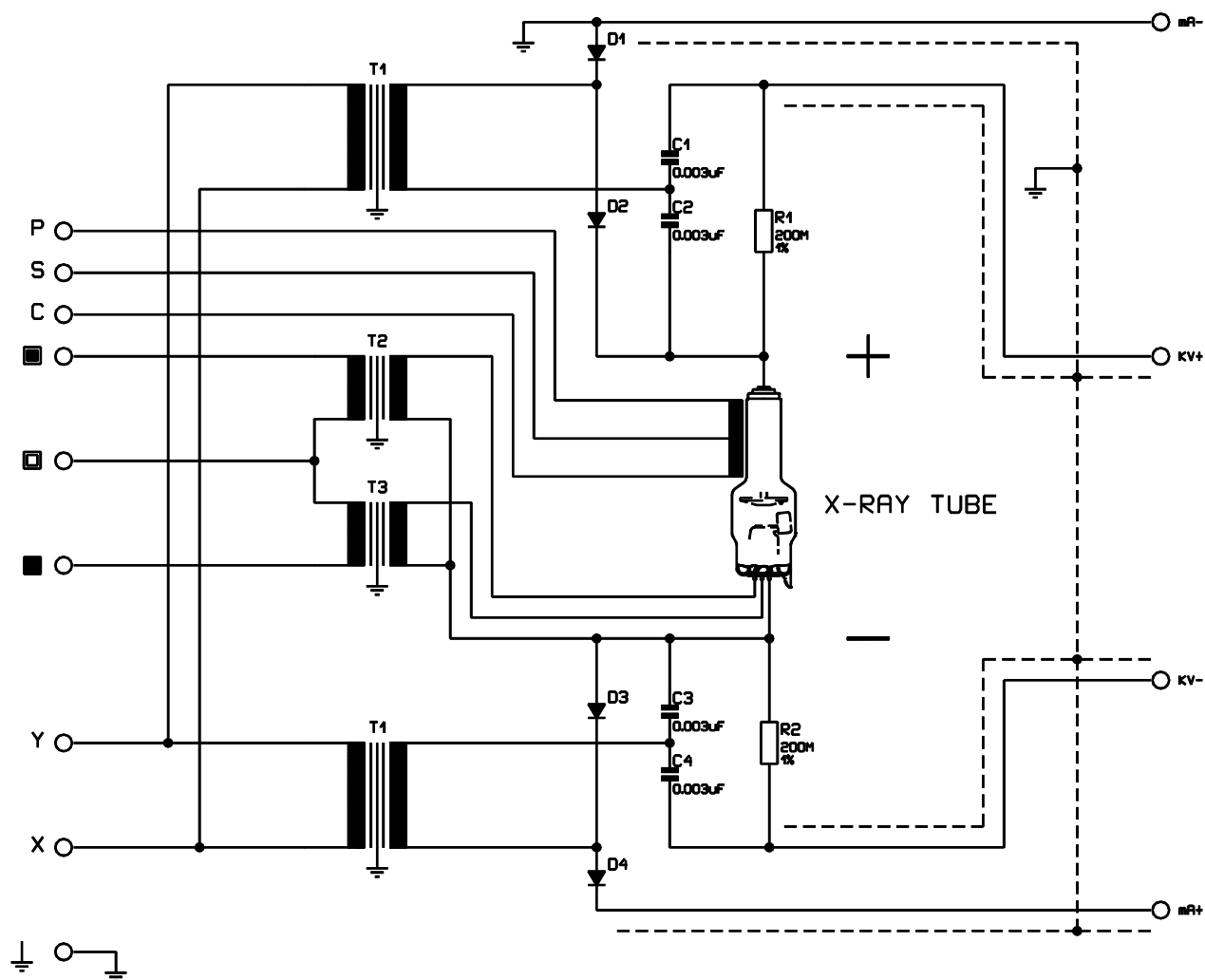
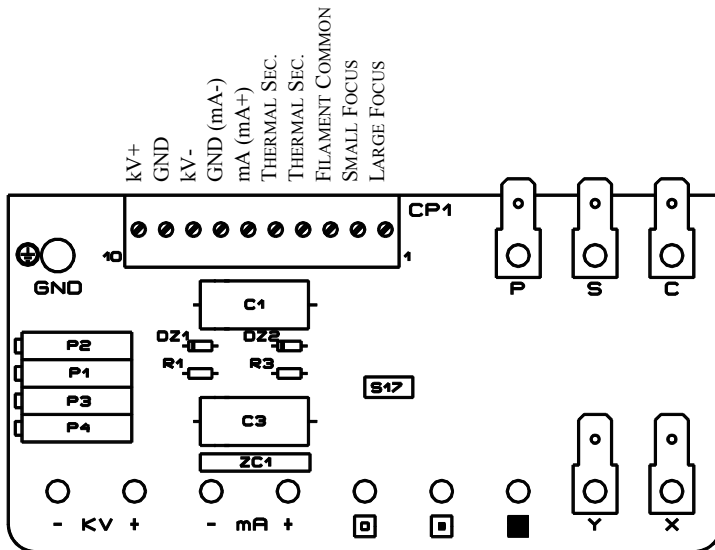
Optionals: 2 disks of aluminium (thickness 1mm), 41/20 o 41/25 lead washer (thickness 2 mm) and seeger.

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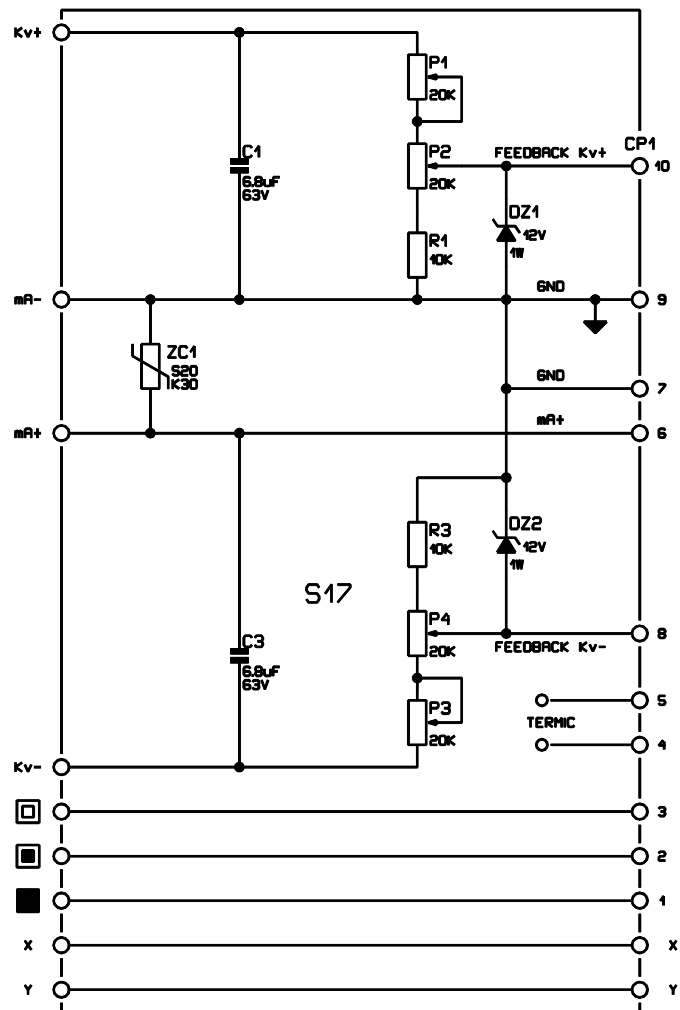Hazards

PROPER USE AND SAFE OPERATING PRACTICES WITH RESPECT TO X-RAY MONOBLOCK GENERATORS ARE RESPONSIBILITY OF EQUIPMENT MANUFACTURERS AND USERS OF SUCH DEVICES. THE MANUFACTURER PROVIDES INFORMATION ON ITS PRODUCTS AND ASSOCIATED HAZARDS, BUT IT ASSUMES NO RESPONSIBILITY FOR AFTER-SALE OPERATING AND SAFETY PRACTICES. LIMITED LIFE, RANDOM FAILURE AND PERFORMANCE REDUCTION ARE INHERENT CHARACTERISTICS OF X-RAY MONOBLOCK GENERATORS. APPROPRIATE SAFE OPERATING PRACTICES SHOULD BE EMPLOYED.

ALL PERSON WHO WORK WITH MONOBLOCK OR EQUIPMENT WHICH UTILIZES SUCH DEVICES MUST TAKE PRECAUTIONS TO PROTECT THEMSELVES AGAINST RADIATION EXPOSURE AND POSSIBLE SERIOUS BODILY INJURY.

DO NOT OPERATE THIS MONOBLOCK EXCEPT IN ACCORDANCE WITH INFORMATION INCLUDED IN THIS TECHNICAL DATA SHEET, THESE PRECAUTIONS, AND ANY ADDITIONAL INFORMATION PROVIDED BY THE MONOBLOCK MANUFACTURER AND/OR COMPETENT SAFETY AUTHORITIES.

X-RAY MONOBLOCK GENERATOR HAZARDS

HIGH VOLTAGE

All X-Ray Monoblock Generators operate at voltages high enough to kill through electrical shock. Equipment designed should prevent personnel contact with high voltages. When direct access to the Monoblock is required, the primary circuits should be disabled and capacitors discharged.

X-RAY RADIATION

This Device produces X-Ray radiation that varies in quantity and energy level, depending upon the voltage and current used when energizing it. Refer to the operating instructions for the equipment. Operating personnel must be protected by appropriate shielding. X-Ray Caution signs or labels must be permanently attached to equipment directing operating personnel never to operate this device without X-Ray shielding. Shielding should be in accordance with applicable Norms. If there is any doubt as to the adequacy of shielding, an expert in this field should be contacted to perform an X-Ray survey.

STORAGE AND CONSERVATION

To guarantee the maintenance of integrity of the product is advised to use or to conserve the system in environments that respect the following climatic factors us.

Humidity				Temperature				Atmospheric pressure			
Use		Storage		Use		Storage		Use		Storage	
min.	max.	min.	max.	min.	max.	min.	max.	min.	max.	min.	max.
30%	75%	10%	90%	10 °C	40 °C	-20 °C	70 °C	500hPa	1600hPa	500hPa	1600hPa

PRECAUTION SHOULD BE TAKEN IN THE DISPOSAL OF X-RAY MONOBLOCK GENERATORS CONTAINING DIELECTRICAL OIL AND/OR LEAD IN ORDER TO COMPLY WITH ALL APPLICABLE EUROPEAN LAWS AND REGULATIONS. IT IS RECOMMENDED THAT FAILED DEVICES BE RETURNED TO THE MANUFACTURER OR AN APPROPRIATE FACILITY TO ENSURE PROPER HANDLING.

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